

AIR-WATERSPRAY™

B E L L D E N T A L P R O D U C T S



Detail of spray pattern



Improve your restorative capability in remote places

Improve your ability to do restorative procedures by adding chip air and cooling water to your high speed hand pieces with the Air-Waterspray™ system. It is designed to interface with our TravelBELL™ battery powered system. The air and water spray is fully adjustable! the system comes with a 3-way syringe.

- Lightweight, only 3.5 lbs.
- Rugged, padded ballistic nylon case and metal control panel
- Includes 3-way syringe
- Designed specifically to interface with the BatteryBELL™ 30XL battery packs
- Totally self-contained
- Works with a variety of high speed E-type nose cones
- Two-year limited warranty

Better Dentistry Through Innovation



B E L L D E N T A L

To order: 800-920-4478



AIR-WATERSPRAY™

Now you can have chip air and cooling water for your high speed hand piece. The Air-Waterspray™ connects to the BatteryBELL™ 30XL battery pack. Then, whenever the motor turns on, chip air and cooling water flow to the hand piece.

The system includes needle valves so you can adjust the air and water flow to get the perfect spray.

It has its own self-contained water supply. Just fill up the water bottle and connect the system to compressed air and you are ready to go.

The battery pack fits neatly in the pocket on the side of the case.

The system is designed for maximum flexibility. Any compressed air source can be used; a compressor, a gas cylinder, or, what is most popular, a simple bicycle pump. And, it can be used with any nose cone, just connect the air-water tubing.

Includes convenient carrying strap and pressure gauge.

Specifications:	
Size	11.5" x 8" x 6"
Weight	3.5 lbs.
Case	Padded ballistic nylon Convenient carrying strap
Water Filter	10 micron
Pressure Operating: Max: Gauge: Interface:	50 PSI 80 PSI up to 100 PSI Standard tire valve, other interfaces available
Pressure Source	compressor, gas cylinder, bicycle pump
3-Way Syringe	Included
Connections	Quick disconnect fittings
Air-Water Flow Adjustment	Via needle valve
Interfaces	With a variety of nose cones
Warranty	2 year limited

Better Dentistry Through Innovation

3301 W. Hampden Ave. Unit N
Englewood, Colorado, USA 80110

Order line: 800-920-4478
FAX: 303-292-4411
www.belldental.com

